




# PURVIEW® Viewer Tools

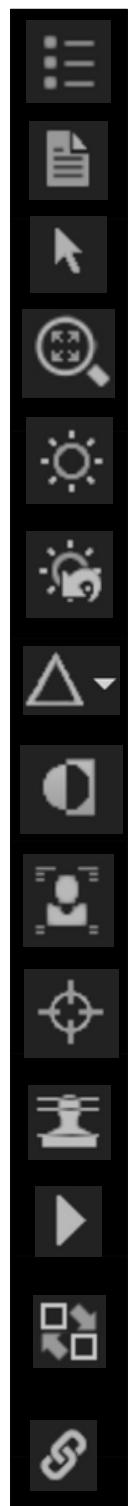


- Report Information-** The report text for the currently selected study appears in this panel
- Study List-** Study list panel presents a related studies list
- Navigate-** Navigate within a series using the left click or cursor keys
- Zoom & Pan-** Zoom images using the scroll wheel, or use the "=" and "-" keys.
- Window & Level-** Adjusts an image's W/L via dragging up & down with the left click. Select CTRL for fine W/L
- Reset W/L-** Resets to the image's default W/L
- Image Sharpening -** Allows access to presets determined at the modality
- Invert-** Invert an image's channels
- Demographics-** Toggle study information on and off
- Triangulation-** Triangulate orthogonal series
- Reference Lines-** Apply guidelines on orthogonal series from the same study
- CINE Tool-** Allows a series of images to be displayed like a movie
- Series Reposition-** Drag and drop series within view ports to other View ports
- Linked Series-** Perform a set of common operations (navigate, zoom, pan, etc)



- Magnifying Glass-** Allows for a closer look at the selected view-port
- Eraser-** Select and erase measurement and markup
- Revert to original-** Reset a series of images to its original state
- Collaboration-** Allows users to share their screen with other participants
- Toggle Markup and Annotation-** Toggle the visibility of overlays across all studies
- Linear Measurement-** When viewing US Echo, measurement, time and velocity are shown.
- Ellipse ROI-** Select the tool then left click and drag the mouse (mm)
- Circle ROI-** Select the tool, then left click and drag the mouse to select your view
- Rectangle ROI-** Select the tool then left click and left click again to end measurement
- Freeform ROI-** Select the tool and then left click.
- Line Annotation-** Draw a line
- Arrow Annotation-** Draw an arrow
- Rectangle Annotation-** Draw a rectangle



- Ellipse Annotation-** Draw an ellipse
- Circle Annotation-** Draw a circle
- Polygon Annotation-** Draw a polygon
- Freehand Annotation-** Allow users to draw a freeform markup
- Text Annotation-** Apply text to an image
- Angle Measurement-** Manually make angle measurements
- Point-** Measures the value of a specific point
- Spine Labeling-** Add specific labels to an image
- Ratio Tool-** Use the tool to draw lines and provide measurements and ratio
- Free Rotate-** Rotate the image by left clicking and dragging the mouse
- Export-** Export an image as JPEG or PNG



- Paper Printing-** Perform a paper print of images on screen
- Select Hanging Protocol-** Choose a hanging protocol to apply to the study
- Screen Layout-** Access study compare or change the layout of each monitor
- Change Rendering-** Adjust slice thickness, final and interactive quality, change MIP, and adjust rendering presets
- Curved MPR-** Select MPR, ADV or 3D view mode, select tool and create curved MPR
- Advanced visualization-** 3D, MPR, Curved MPR, Fusion
- Cut Planes (3D)-** Rotate 3D Volume to desired view and select tool.
- Scalpel (3D)-** Remove or isolate a segment of the image. Drag mouse around desired area, then click region to keep. Reset will revert all segments.

**Need Help? Contact us:**  
**+1 (800) 501-1537 or [support@purview.net](mailto:support@purview.net)**  
To learn more, visit us at [www.purview.net](http://www.purview.net)